Lansinoh®

Manual Breast Pump

INSTRUCTIONS FOR USE

Lansinoh.

Manual Breast Pump

Congratulations! Your decision to provide your baby with breastmilk, even if you need to be apart, is one of the most precious gifts you can give your baby. Lansinoh's Manual Pump is a safe and effective choice for moms who need to pump occasionally and who appreciate the convenience of a manual pump.

CONTENTS

1. Introducing the Lansinoh® Manual Breast Pump 1
2. Lansinoh® Manual Breast Pump Parts..... 1
3. How to Clean and Sanitize Your Lansinoh® Manual Breast Pump. . . . 2
4. How to Assemble Your Lansinoh® Manual Breast Pump. 2
5. How to Use Your Lansinoh® Manual Breast Pump 4
6. Breastmilk Storage Guidelines. 5
7. Tips About Pumping 5
8. Troubleshooting for the Lansinoh® Manual Breast Pump. 6

Intended Use of Breast Pump

Lansinoh® Manual Breast Pump is intended to express and collect the breastmilk of a nursing woman for the purpose of feeding collected breastmilk to a baby. Lansinoh® Manual Breast Pump is intended for a single user.

Indications of Breast Pump

Lansinoh® Manual Breast Pumps are indicated to eliminate clinical situations such as milk stasis in the breast, engorgement and mastitis caused by insufficient milk discharge, relieve sore and cracked nipples and to bring out flat or inverted nipples. Also, Lansinoh® Manual Breast Pumps enable mothers to provide breast milk for babies who cannot feed directly at the breast for a variety of reasons, such as latch-on difficulties, premature or unwell babies.

Contraindication of Breast Pump

There are no known contraindications.

IMPORTANT SAFEGUARDS

The Manual Breast Pump is a personal care item and for hygienic reasons should never be shared between mothers or resold from one mother to another. If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breastmilk using the Lansinoh Manual Pump will not reduce or remove the risk of transmitting the virus to your baby through your breastmilk.

Save These Instructions

along with your receipt for future reference.

PLEASE DO NOT RETURN THIS PUMP TO THE RETAILER.


Lansinoh's dedicated Customer Service team is here to help. If you experience any issues with your Lansinoh® Manual Pump please call **800-292-4794** or email customerservice@lansinoh.com.

1. Introducing the Lansinoh® Manual Breast Pump

The Lansinoh® Manual Breast Pump has been designed specifically with the needs of breastfeeding mothers in mind. This breast pump has two phases: Let-Down Phase and Expression Phase. You can easily switch between the two phases by adjusting the position of the handle on the stem. The ComfortFit® flange creates a seal providing reliable suction resulting in comfortable and fast milk flow (standard and large size included). The wide neck bottle is interchangeable with the entire Lansinoh® pump range. The pump is easy to assemble, easy to use and easy to clean.

2. Lansinoh® Manual Breast Pump Parts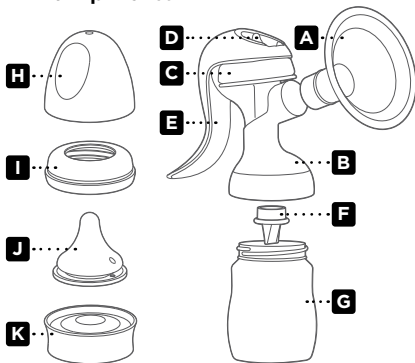

	NAME	ITEM NUMBER
A	ComfortFit® Flange (2x)	53408 (25mm) 50428 (30.5mm)
B	Manual Pump Body	50631
C	Silicone Diaphragm	50626
D	Manual Pump Stem	50627
E	Manual Pump Handle	50621
F	White Valve (2x)	53400
G	Collection Bottle	53562
H	Bottle Cap	53462
I	Bottle Collar	53463
J	NaturalWave® Nipple	71027
K	Bottle Storage Lid	53564

3. How to Clean and Sanitize Your Lansinoh® Manual Breast Pump

Sanitize:

Prior to using your pump for the first time, disassemble it (remove silicone diaphragm and stem, ComfortFit® flange and white valve) and boil all parts for 5 minutes. Remove all parts from water immediately after boiling. Completely air dry before assembly and before the next use.

Note: It is essential that you use enough water to keep the parts freely floating while boiling.

Cleaning:


The breast flange, bottle, and handle may be cleaned in the upper compartment of your dishwasher or hand washed with dish soap and hot water. Do not use solvents or abrasives. Rinse with hot, clean water. The white valve, silicone diaphragm, stem, NaturalWave® nipple, and the ComfortFit® flange can be washed in the dishwasher; however, we recommend washing them by hand to prevent loss and to extend their usefulness. Completely air-dry all parts on a clean towel or drying rack.

Note: Use caution when cleaning the small white valve. It is important for proper suction. Do not use excessive force and do not clean with a baby bottle brush or a nipple brush. Gently wash by hand in hot, soapy water and rinse under hot running water.


4. How to Assemble Your Lansinoh® Manual Breast Pump

NOTE: Prior to assembling your Lansinoh® Manual Breast Pump, ensure that all parts have been cleaned and sanitized as detailed in section 3 of this manual.


1. Wash your hands thoroughly.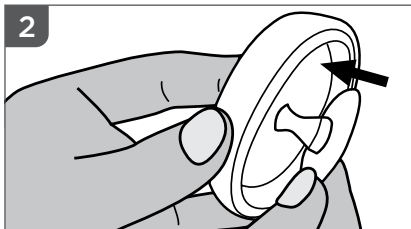


2. Insert the stem into the silicone diaphragm.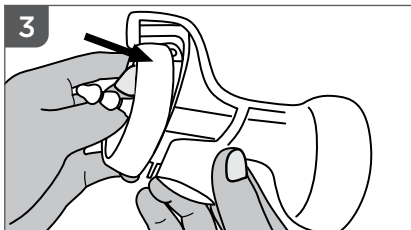


3. Place the silicone diaphragm and stem on top of the pump body. Make sure it fits securely around the rim by pressing down with your fingers to ensure a perfect seal.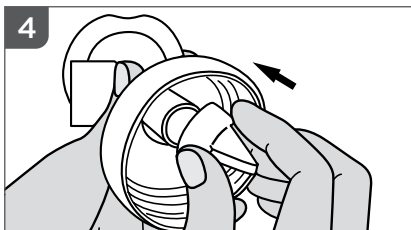


4. Gently secure the white valve onto the pump from underneath, making sure the white valve is flush with the pump body. It is not necessary to use excess force when applying the white valve. If you find that the white valve is difficult to remove, you have pushed it too far onto the pump body.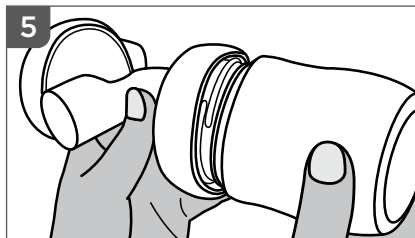


5. Screw the bottle onto the pump body.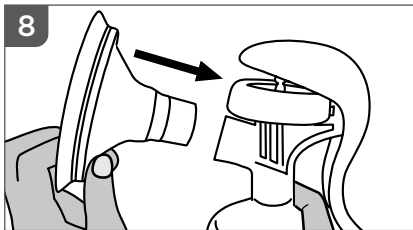

8. Select the flange that fits best. Attach the ComfortFit® Flange to the flange body.

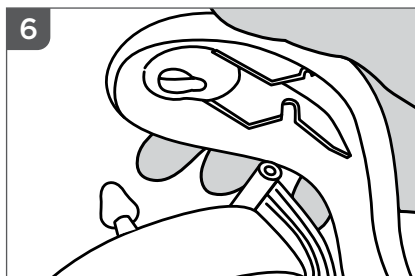
6. Align the groove in the handle with the stem and gently press the handle down. Push until you hear it click.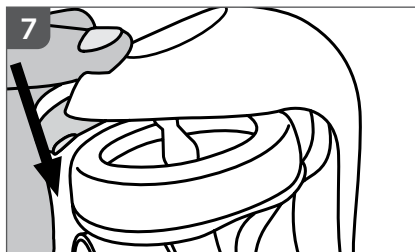

7. Adjust the stem and handle so that the handle fits the first notch of the stem, known as the Let-Down Phase.*

**This breast pump has two separate phases, Let-Down Phase and Expression Phase. You can easily switch between the two phases by adjusting the position of the handle on the stem.*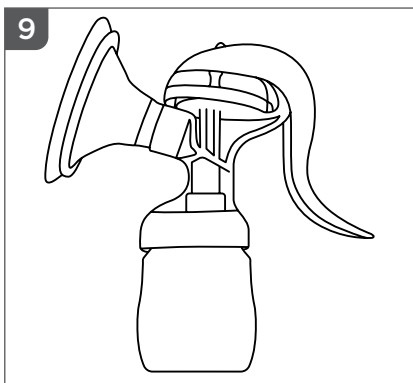

9. When fully assembled, this is what your pump should look like.

5. How to Use Your Lansinoh® Manual Breast Pump

If possible, wait to begin pumping until your breastmilk supply and breastfeeding schedule are well established. This is normally at least 3-4 weeks after birth, unless told otherwise by your doctor or breastfeeding advisor.

1. Wash your hands thoroughly.
2. Make sure you have sanitized and carefully assembled your pump exactly as described in the sections "How to Clean and Sanitize Your Lansinoh® Manual Breast Pump" and "How to Assemble Your Lansinoh® Manual Breast Pump."
3. Center the pump opening over your nipple and gently press it against your breast ensuring that your nipple is centered in the tunnel of the flange. Press the ComfortFit® Flange of the pump firmly against your breast making sure the seal is secure so that no air can escape. You can create a good seal by using your other hand to support your breast from underneath.

If your nipple is unable to be centered within the tunnel of the flange without rubbing, you may need the 30.5mm ComfortFit® flange included in the pack.

4. As you gently push down on the pump handle you will feel the suction on your breast. You do not need to depress the handle fully to create suction, only as much as is comfortable.
5. You can create gentle suction pressure in preparation for expressing by setting the handle in the first (top) notch of the stem. Confirm that your nipple is moving and squeeze and release the handle until you find your preferred pace.

Warning: Do not continue pumping for more than five minutes at a time if no result is achieved. Try expressing at another time during the day.


Please remember: Should pumping become very painful or uncomfortable, you should stop using the pump and consult your doctor or breastfeeding advisor.

Using Lansinoh's 2-Phase Technology

This breast pump has two phases, Let-Down Phase and Expression Phase. You can easily switch between the two phases by adjusting the position of the handle on the stem.

Let-Down Phase

This phase gently stimulates the breast. It is designed to mimic the way a baby feeds at the beginning of a feeding. It is one of the features that encourages milk let-down and encourages your milk to begin to flow. You can create gentle suction pressure by setting the handle in the first (top) notch in the stem and using quick pulses with the handle.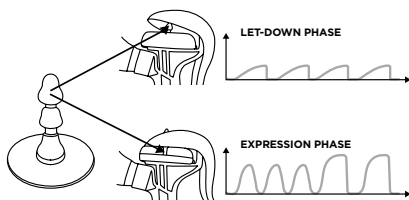


Expression Phase

This phase allows you to express breastmilk from the breast. Switch to this phase by setting the handle in the second (bottom) notch in the stem. To get the strongest suction fully depress the handle. For reduced suction strength depress the handle less.

To start with the Let-Down Phase, set the stem in the first (top) notch of the handle and squeeze the handle. Your breastmilk will soon start flowing even though you may not use all the suction the pump can generate.


Once breastmilk is flowing (1 - 2 minutes) change to Expression Phase.

To switch to Expression Phase, gently press the "PUSH" part at the tip of the handle to set the handle to the second (bottom) notch of the stem.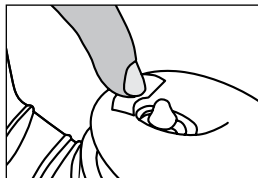

Continue to squeeze and release the handle to find the appropriate suction pressure.

If the suction pressure feels weak when expressing your breastmilk, squeeze the handle all the way. The suction pressure changes based on how hard you squeeze the handle. Expressing in comfort is more important than setting a strong suction pressure. Please express milk at a suction pressure that feels appropriate for you.

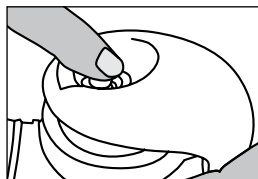*To return to Let-Down Phase from Expression Phase, (1) Pull the tip of the stem towards you, (2) gently squeeze the handle, and set it in the first (top) notch.

6. Breastmilk Storage Guidelines

Breastmilk Storage and Thawing Guidelines for Full-term Healthy Babies

Check with your healthcare provider or an International Board Certified Lactation Consultant for specific storage instructions. When freezing, store in the back of the freezer and do not store milk in the door as temperature can fluctuate as the door is opened and closed.

Guide to storing fresh breastmilk for use with healthy, full-term babies

PLACE	TEMPERATURE	HOW LONG	THINGS TO KNOW
Countertop, table	Room temperature (up to 77°F)	Up to 4 hours	Containers should be covered and kept as cool as possible. Covering the container with a clean cool towel may keep milk cooler. Throw out any leftover milk within 2 hours after the baby is finished feeding.
Refrigerator	40°F	Up to 4 days	Store milk in the back of the refrigerator. When at work, it's OK to put breastmilk in a shared refrigerator. Be sure to label the container clearly.
Freezer	0°F or colder	Within 6 months is best. Up to 12 months is acceptable	Store milk toward the back of the freezer where the temperature is most constant. Milk stored at 0°F or colder is safe for longer durations, but the quality of the milk might not be as high.

Source: Adapted from 7th Edition American Academy of Pediatrics (AAP) *Pediatric Nutrition Handbook* (2014); 2nd Edition AAP/American College of Obstetricians and Gynecologists (ACOG) *Breastfeeding Handbook for Physicians* (2014); Academy of Breastfeeding Medicine (ABM) *Clinical Protocol #8 Human Milk Storage Information for Home Use for Full-Term Infants* (link is external) (2017); CDC *Human Milk Storage Guidelines* (2018).

Date all breastmilk and use the oldest breastmilk first.

Thawing

Thaw frozen milk in the refrigerator or place the containers under cool running water, gradually warming the water until the milk is thawed. Thawed milk can be kept in a refrigerator for up to 24 hours.

NOTE: DO NOT refreeze milk once it is thawed

Gently swirl thawed milk before feeding to baby to mix the layers that have separated. Breastmilk is not homogenized and the "cream" will rise to the top of the bottle. Breastmilk can vary in color and consistency depending upon the time of the day that the milk was expressed, what you have eaten, and the age of the baby at the time of pumping.

NOTE: DO NOT MICROWAVE HUMAN MILK!

Microwaving alters the composition of the milk. Microwaving also may create "hot spots" in the milk that could burn the infant.

NOTE: DO NOT thaw breastmilk in hot or boiling water.

7. Tips About Pumping

- Familiarize yourself with the pump and parts prior to use.
- The most important part of successful pumping is the let-down reflex. This is when your breastmilk begins to flow freely. Warm wet compresses applied to the breasts and gentle, circular massaging of the breasts prior to and/or during a pumping session can help encourage this natural reflex.
- Try to choose a time to pump when you are not rushed and will not be interrupted. Trying to relax and thinking of your baby or looking at a photo of your baby can also be helpful when pumping your breastmilk.
- Overall a pumping session can last about 20-30 minutes but it depends on your body. Remember, successful pumping is a learned art. Early practice sessions may be shorter or longer than stated here and may result in a small amount of breastmilk collected, but that does not mean your body is not making enough milk or that you are doing anything wrong. Milk is produced on a supply and demand basis and pumping your breasts regularly in addition to nursing your baby stimulates and maintains your breastmilk supply.
- If breastfeeding problems occur, call your physician, an International Board Certified Lactation Consultant, or breastfeeding counselor for an in-person consultation.
- You may pump one breast while nursing the baby on the other. This allows you to maximize pumping effectiveness using the let-down reflex stimulated by your baby's suckling.
- Make sure the tunnel of the breast flange is tilted slightly downward to allow your breastmilk to flow naturally toward the bottle.
- Do not turn the pump upside down after pumping.

Precautions when feeding expressed breastmilk:

- DO NOT MICROWAVE BREASTMILK! Microwaving alters the composition of the milk and may also create "hot spots" in the milk that could burn the infant.
- Always test the temperature of any liquid before feeding your baby.
- Do not let a child take a bottle to bed or self feed.
- To prevent possible choking hazard, test strength of the nipple by pulling on the bulb portion. Discontinue use if any tear or crack appears.

8. Troubleshooting for the Lansinoh® Manual Breast Pump

Lack of Suction / Decreased Suction

If it feels like the suction has decreased, try the following:

- Make sure that all pump parts have been properly dried and properly assembled.
- Inspect the white valve for a tear or pinhole. This component is critical to achieving proper suction. If necessary, replace the white valve with the spare valve provided with the pump.
- Ensure that the silicone diaphragm and stem are fitted securely to the pump body and a perfect seal is created.
- Ensure that the ComfortFit® Flange is fitted securely to the flange body and a perfect seal is created.
- If your nipple is unable to be centered within the tunnel of the flange without rubbing, you may need the 30.5mm ComfortFit® flange included in the pack.

No Breastmilk Expressed

If you are not successful in expressing any milk, try the following:

- Make sure that all the parts have been properly dried and assembled and that proper suction is created.
- Try to relax as successful pumping is a learned art and may take several practice attempts to get started. Gentle breast massage before and during pumping can also aid in milk let-down and flow.
- Please refer to the "Tips About Pumping" section.

Pain When Pumping

If you are experiencing pain when expressing breastmilk, try the following:

- You may be pumping too hard. It may not be necessary to use all of the suction that the pump can generate. Try depressing the handle half way.
- You may need a larger or smaller flange. Try the other flange included in this package. Consult your lactation consultant for an assessment and sizing.
- Consult your physician, breastfeeding advisor or lactation consultant.

Lost or Broken Parts

Spare parts are available through Lansinoh® Customer Service at **800-292-4794** or on our website at lansinoh.com/store.

If you are still experiencing problems, please contact Lansinoh® Customer Service at **800-292-4794** or contact your physician or breastfeeding advisor.

SYMBOLS	MEANING OF THE SYMBOLS
	Read and follow the instructions for use
	Identifies the manufacturer
	Indicates the date of manufacture
	Indicates the part number of the device
	Indicates the batch number
	Keep the device away from sunlight
	Keep the device dry
	Indicates that the packaging contains products intended to come in contact with food
	Indicates to handle the fragile device with care

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This product is a personal care product and therefore cannot be returned once opened. If you experience any problems with this product, please telephone our toll free customer service line at 800-292-4794. Our representatives will be happy to assist you.

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Lansinoh® is a registered trademark of Lansinoh Laboratories, Inc.
© 2023 Lansinoh Laboratories, Inc. All rights reserved. Printed in Türkiye

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Manufacturer: / Fabricante: Lansinoh® Laboratories Sağlık Gereçleri Tasarım San. Tic. Ltd. Sti.
Lansinoh Laboratories, Inc, Alexandria, VA 22314 USA, 800-292-4794 | Lansinoh.com
Lansinoh® es una marca registrada de Lansinoh Laboratories, Inc.
© 2023 Lansinoh Laboratories, Inc. Reservados todos los derechos. Impreso en la República de Turquía