LatchAssist[®] Nipple Everter

For best results

Have your baby positioned for breastfeeding and then use LatchAssist[®] nipple everter. This will enable your baby to latch onto your breast as soon as your nipple is extended. The LatchAssist[®] nipple everter comes in two sizes so you can choose the right fit for you. A case for hygienic storage and on-the-go use is also included.

Indications for use


To help the correction of inverted or flattened nipples of the postpartum lactating mother, by everting the nipple, allowing an infant to latch.

Sanitizing before first use

We recommend sanitizing the LatchAssist[®] prior to first use and regularly as needed during use. The LatchAssist[®] can be sanitized by disassembling and boiling all parts in water for five minutes. Please be sure that no parts or attachments touch the bottom or the sides of the pan as this could cause them to melt.

Directions for use

- Always wash and dry hands thoroughly with soap and water before handling the LatchAssist[®]
- Select the LatchAssist[®] flange size that is most comfortable for you – either the 19mm or 24mm
- How do you know which size is the best fit? If your nipple rubs along the flange tunnel, it's too small. If there is too much room around the nipple in the flange, it's too large.
- Using a clean LatchAssist[®], gently squeeze the bulb and place the shield over and around your nipple

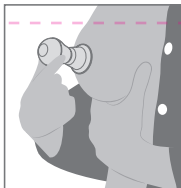- Lightly press the shield down against the skin to ensure an airtight seal
- Slowly release the bulb for the desired pressure
- Repeat gently until your nipple is extended and the baby can easily latch onto the breast
- To remove, squeeze bulb and gently pull away from breast
- After use, discard any milk that may have collected in the bulb, then wash, rinse and air dry

Cleaning after use

After each use, rinse all the LatchAssist[®] parts with cold water immediately.

LatchAssist[®] may be washed in a dish washer.

If washing by hand:

- Place the parts in a clean wash basin used only for washing infant feeding equipment. Do not place the parts directly in the sink, because germs in sinks or drains could contaminate the parts.
- Hand wash the parts using warm soapy water.
- Rinse the parts with running clean, warm water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
- Place the parts on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry. Allow the parts to dry completely before assembly and use.
- Once the parts are dry, they can be stored in the clean case.
- Clean the wash basin and the nipple brush after each use following the manufacturers recommendation for cleaning. Rinse them well and allow them to air-dry after each use.

Distributed by:

Lansinoh Laboratories, Inc. Alexandria, VA 22314, USA
+1 800-292-4794 | Lansinoh.com

Made in Malaysia

Lansinoh[®] and LatchAssist[®] are registered trademarks of Lansinoh Laboratories, Inc. © 2024 Lansinoh Laboratories, Inc. All rights reserved.

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