Contact Nipple Shields

Lansinoh® Contact Nipple Shields can be effective short-term tools for addressing latch-on issues but should not be used until milk supply is established and should not be used within the first week after birth. It is important that the underlying causes of early challenges are assessed by a lactation professional and that nipple shields are only used under the guidance of this professional.

Lansinoh® Contact Nipple Shields can be used to address issues related to:

- **Premature, ill, or small infant.** May make feeding easier for a small or weak baby because suction inside the nipple shield holds the nipple in an everted position. Milk pools in the tip of the shield and provides an immediate reward when the baby resumes sucking.
- **Flat or inverted nipples.** Some new mothers have nipple tissue that does not stretch easily, making it difficult for baby to draw in the mother's nipple. As a result, the baby may pull away, cry, or simply fall asleep. Lansinoh® Contact Nipple Shields provide sensation deep in the mouth to encourage baby to keep sucking. This continued sucking encourages the nipple to elongate and eventually nipple shields may not be needed.
- **Tongue and/or lip tie.** A small percentage of babies are born with tongue / lip tie. In these cases, connective tissue prevents baby from fully extending their tongue or flanging their lips, and this makes it difficult for baby to latch properly. Lansinoh® Contact Nipple Shields allow baby to feed using a shallower latch until the tongue / lip tie can be resolved by a physician.
- **Overactive let-down.** Some mothers may experience a particularly forceful let-down reflex. This can cause baby to choke, sputter, or pull off the breast when feeding. Using a nipple shield can help slow the flow of milk. Once baby has become more experienced at breastfeeding, they will be able to regulate the flow of milk without the nipple shield in place.

Nipple Shields allow moms to continue breastfeeding while these short term latch-on issues are addressed and resolved. Nipple shields should be used only

until the situation is corrected, so it is important to address the underlying cause of breastfeeding issues. Weaning to the bare breast is the optimal goal.

WARNING: Nipple shields should be used under the direction of a lactation professional. They should not be relied upon for long term use. Nipple shields should not be used in the first week after birth unless specifically advised by a lactation professional, as their use may affect milk supply. Make sure that your nipple shield is the proper fit to ensure good milk transfer and avoid nipple damage.

Cleaning & Care: Always wash and dry hands thoroughly with soap and water before handling nipple shields.

Before first use:

- Sanitize the nipple shields by boiling in water for 5 minutes.
- Put the shields in a pot and cover with water, making sure there is room for them to float freely in the pot. Bring the water to a boil and allow to boil for 5 minutes.
- After boiling, remove from the pot with tongs and place on a clean towel to dry, covering the shields with a second towel.
- Allow to dry completely before use.
- Once the nipple shields are dry, the shields can be stored in the case with the nipple shields facing up. Do not store the shields in the case until fully dry.

Before and after each use:

- Rinse the nipple shield with cold water and place the shield in a clean wash basin used only for washing infant feeding equipment. Do not place shields directly in the sink, because germs in sinks or drains could contaminate the shield.
- Hand wash the shield using warm soapy water.
- Rinse with running clean, warm water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
- Place the clean nipple shield on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry.
- Allow shields to dry completely before use.
- Once the shield is dry, it can be stored in the protective case with the nipple shield facing up. Do not store in the case until fully dry.
- Clean the wash basin after each use following the manufacturer's recommendation for cleaning. Rinse well and allow to air-dry after each use.

- Regularly wash the protective case following the instructions for the shields. Dry thoroughly before storing the nipple shields.

Directions for Use:

- Wash your hands with soap and water and rinse thoroughly before applying the shield to your breast.
- The cut out area of the nipple shield should be placed where the baby's nose will be when feeding.
- Gently pull the sides of the shield back, turning the shield partially inside out.
- Bring the shield to your breast, lining your nipple up with the shield's nipple opening.
- Push the rim of the shield toward your breast, enclosing as much of your nipple as possible in the nipple opening.

Weaning from Nipple Shields: Nipple shields are intended for short term use. Once the underlying issue causing your challenge has been addressed, it is best to wean from nipple shields. Depending on the type of issue being addressed, baby may only need to use the shields for a few sessions. Others may take longer to resolve. Be sure to consult with a lactation professional regarding use of nipple shields and weaning off of them.

- A few minutes prior to feeding, hand express or pump until a let-down occurs. This will help elongate the nipples and baby will be able to get milk as soon as they latch on.
- Offer baby the breast without the nipple shield. Often once the underlying issue is resolved baby is able to latch easily.
- If baby is unable to latch, place the shield on the breast and allow baby to nurse for a minute or two. Then unlatch, remove the shield, and resume nursing.
- If baby becomes upset while trying to latch on, stop and attempt to calm baby before attempting to latch again.

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