


# Customize your pumping sessions

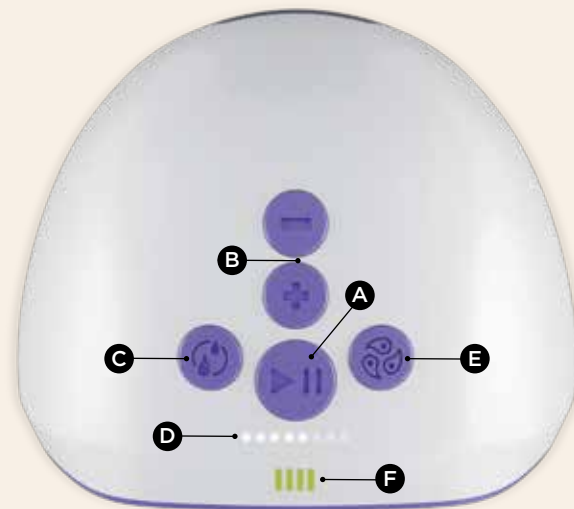

## Pump Controls

- A. Power - On/Off/Pause/Resume
- B. Suction Level Control
- C. Let-Down/Expression Toggle Button
- D. Suction Level Indicator Lights - From 1 to 8
- E. Pumping Style Toggle Button
- F. Battery Level Indicator Lights

Battery Level Indicator Lights	
4 GREEN BARS	Enough charge for approximately 8 sessions
3 GREEN BARS	Enough charge for approximately 6 sessions
2 GREEN BARS	Enough charge for approximately 4 sessions
1 GREEN BAR	Enough charge for one full pumping session
1 RED BAR	Battery low, inadequate charge for pumping session
FLASHING RED LIGHT	Pump will turn off

\*A session time is approximated at 15-20 minutes, but may vary by individual

Charge before your first pumping session. Green battery lights will flash when charging is in progress and 4 green bars will illuminate when your pump is fully charged.  
**Note:** The pump will not turn on during charging.



# Questions?

Our pumping experts are here to help.



Email us at [customerservice@lansinoh.com](mailto:customerservice@lansinoh.com)



Call us at 1-800-292-4794



Chat at [Lansinoh.com](https://www.lansinoh.com)

We are *with* you.

WP53750QS0324



Sign up for exclusive access

Join our village for pump support and exclusive access to:

Hassle-free replacement parts

Monthly product giveaways

Product offers, promotions & more



Wearable Pump Quick Start Guide

# Lansinoh®

## Welcome, Mama

Pumping freedom is at your fingertips



# Meet your new favorite pump



Before you get started, disassemble, clean, and disinfect your pump components according to instructions in Section 2 of the Instructions for Use manual. Be sure to wash and dry hands before handling your Wearable Pump.

# Quick and easy assembly



**IMPORTANT!** Make sure your nipples are centered in the flange tunnel before pumping. Use the flanges plus inserts to find your best fit. For more information on flange sizing, see Section 4 in the Instructions for Use manual.

**IMPORTANT!** Ensure cups are completely connected with a tight seal to prevent milk leakage. Do not disassemble the pumping cups until you have poured out the milk.