Lansinoh.

## Lansinoh® ComfortFit® Flanges Instruction For Use

**Lansinoh® ComfortFit® Flanges** were designed for use with Lansinoh breast pumps. For best results, use these flanges only with Lansinoh breast pumps.

### Cleaning Instructions

Unless otherwise instructed by your healthcare provider, follow these sanitizing and cleaning instructions:

#### Before First Use and Daily Sanitization

1. Prior to using the set for the first time, boil for 5 minutes. Ensure you use enough water to keep flanges afloat while boiling.
2. After boiling remove from the pot with tongs and place on a clean towel or paper towel to dry. Covering the flanges with a second towel.
3. Allow the flanges to dry completely before assembly and use.

#### After Each Use

1. Rinse the flanges with cold water immediately after each use. Place the flanges in a clean wash basin used only for washing infant feeding equipments. Do not place directly in the sink, because germs in sinks or drains could contaminate the flanges.
2. Wash the flanges using warm soapy water.
3. Rinse with running clean, warm water or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding equipments.
4. Place the flanges on a clean towel or paper towel to dry in an area protected from dirt and dust. **Do not** use a dish towel or paper towel to rub or pat the flanges dry.
5. Once the flanges are dry they can be stored in a clean plastic bin with a lid or in a clean plastic zipper bag.
6. **Do not** use disinfectants for cleaning flanges.

#### Before Each Use

1. Always wash hands thoroughly with soap and water before handling the flanges, breastmilk expression set, the pump body, or other attachments.
2. Visually inspect the individual components for cracks, chips, tears, discoloration or deterioration, as this may affect the functionality of the breastmilk expression set. Please call **1-800-292-4794** or email [customerservice@lansinoh.com](mailto:customerservice@lansinoh.com) for further guidance and replacement parts.

### Assembly

Firmly push **ComfortFit® Flange** into breast flange body as shown:

### Sizing

Please refer to the sizing information below to ensure proper flange fit:

A proper fit is essential for a mom's comfort and maximum milk flow. An improperly fitting flange can cause discomfort and impact milk let-down and flow.

#### Correct fit

Nipple is easily drawn into tunnel and moves freely.

#### Too small

Nipple rubs along flange tunnel.

#### Too large

Too much room around nipple in the flange.

If you feel any discomfort, you may need a different size, please visit [www.Lansinoh.com](http://www.Lansinoh.com).

Save these instructions along with your receipt for future reference.

Distributed exclusively by:  
Lansinoh Laboratories, Inc.  
Alexandria, VA 22314 USA  
+1 800-292-4794  
[Lansinoh.com](http://Lansinoh.com)

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DE53510IF0820

ComfortFit® is a U.S.  
registered trademark  
by Lansinoh

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Saglik Gereçleri Tasarım San. Tic. Ltd. Sti.