



Product is not made with natural rubber latex




Does not contain phthalate plasticizers DEHP, DiBP, DBP, or BBP



Does not contain BPA, BPS

Primo-Lacto[®] - Colostrum Collector

A Closed System for Colostrum Collection - Medela[®] Breast Pump Kit, ENFit[®] Compatible

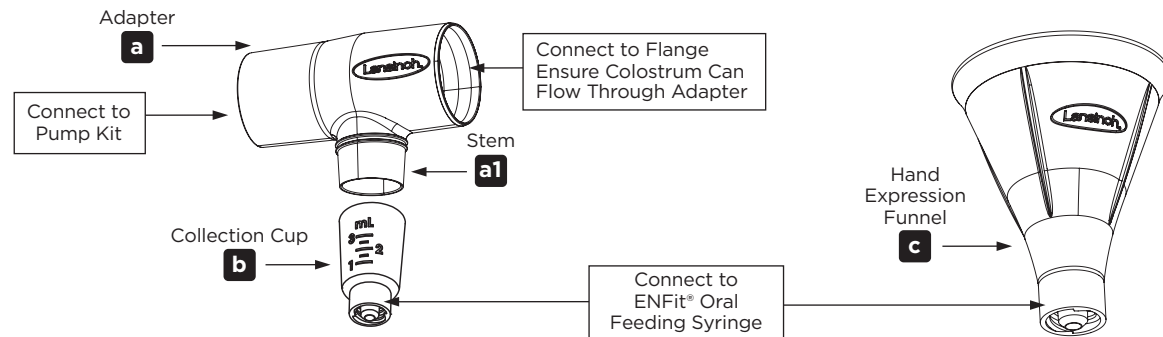


To use while pumping:

1. Wash hands with soap and water before each use.
2. Connect the adapter (**a**) to the pump kit. Press together to ensure a secure fit.
3. Connect the adapter (**a**) to the pump flange. Press together to ensure a secure fit.
4. Screw a collection cup (**b**) onto the adapter stem (**a1**).
5. Attach bottle to pump kit per manufacturer instructions.
6. Screw an ENFit[®] compatible oral feeding syringe into the bottom of the collection cup (**b**). Ensure a tight fit. Ensure syringe plunger is all the way up.
7. Begin pumping session. The colostrum will flow directly into the collection cup (**b**).
8. At the end of the pumping session, turn off pump and remove flange from breast. Pull the syringe plunger down to vacuum the expressed colostrum from collection cup (**b**) into the syringe. Remove the syringe.
9. Cap and label the syringe. Follow your healthcare provider's instructions for feeding or storage.

Primo-Lacto® - Colostrum Collector

To use while hand expressing:

1. Wash hands with soap and water before each use.
2. Screw an ENFit® oral feeding syringe into the bottom of the hand expression funnel. Ensure a tight fit. Ensure syringe plunger is all the way up.
3. Begin hand expression and collect the expressed colostrum with the hand expression funnel.
4. At the end of the session, pull the syringe plunger down to vacuum the expressed colostrum from the hand expression funnel into the syringe. Remove the syringe.
5. Cap and label the syringe. Follow your healthcare provider's instructions for feeding and storage.

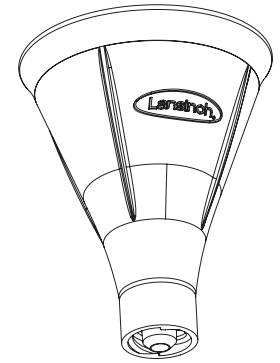

Hand
Expression
Funnel

⚠ CAUTION!

Cleaning: Wash with soap and hot water between sessions, rinse with clean water and air dry.

| | Primo-Lacto® |
|----------------|--|
| Before 1st Use | ✓ Sterile - no additional cleaning necessary |
| After Each Use | ✓ Wash - use soap and hot water |
| Once Per Day* | ✓ Sanitize - boil for 5 minutes |

*Only necessary if directed by health care provider

For hospital use only during the colostrum stage of lactogenesis. Kit is designed for single patient use only. Components may be used multiple times. Inspect components prior to each use for damage or cracks. If required, another kit may be used.

Lansinoh® and Primo-Lacto® are registered trademarks of Lansinoh Laboratories, Inc. ENFit® is a registered trademark of Global Enteral Device Supplier Association, Inc. Medela® is registered trademark of Medela Holding AG Corporation.
© 2024 Lansinoh Laboratories, Inc. All rights reserved. Made in China
Distributed by Lansinoh Laboratories, Inc. Alexandria, VA 22314, +1 800-292-4794
For further information, visit www.primolacto.com



Lansinoh Laboratories
Sağlık Gereçleri Tasarım San. Tic. Ltd. Sti.
Zafer SB Mah. Gunduz Sk. No:13 E.S.B
Gazimirmir / Izmir / Türkiye

REF 59000
Catalog number

LOT
Batch code


Date of
Manufacture


Use by date

STERILE EO
Sterilized using
ethylene oxide


Consult
instructions
for use


Manufacturer
name and
address


Keep away
from sunlight


Keep dry


Do not use if
package is
damaged.


Do not
re-sterilize

UDI
Unique device
identifier

PL59000IF1023