



Primo-Lacto® - Hand Expression Kit

Slip Tip Compatible

To use while hand expressing:

1. Wash hands with soap and water before each use.
2. Press a Slip Tip compatible oral feeding syringe into the bottom of the hand expression funnel. Ensure a tight fit. Ensure syringe plunger is all the way up.
3. Begin hand expression and collect the expressed colostrum with the hand expression funnel.
4. At the end of the session, slowly pull the the syringe plunger down from the hand expression funnel to vacuum the expressed colostrum into the syringe. **IMPORTANT:** Remove the syringe by carefully rotating it from the hand expression funnel.
5. Cap and Label the syringe. Follow your healthcare provider's instructions for feeding and storage.

⚠ CAUTION!

Cleaning: Wash with soap and hot water between sessions, rinse with clean water and air dry.

	Primo-Lacto®
Before 1st Use	✓ Sterile - no additional cleaning necessary
After Each Use	✓ Wash - use soap and hot water
Once Per Day*	✓ Sanitize - boil for 5 minutes

*Only necessary if directed by health care provider

For hospital use only during the colostrual stage of lactogenesis.
Kit is designed for single patient use only. Components may be used multiple times. Inspect components prior to each use for damage or cracks. If required, another kit may be used.




Product is not made with natural rubber latex



Does not contain phthalate plasticizers DEHP, DIBP, DBP, or BBP



Does not contain BPA, BPS

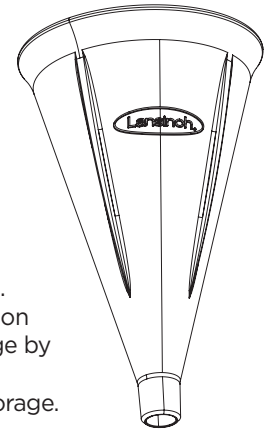

Hand Expression Funnel

Lansinoh® and Primo-Lacto® are registered trademarks of Lansinoh Laboratories, Inc.

© 2024 Lansinoh Laboratories, Inc
All rights reserved. Made in China

Distributed by Lansinoh Laboratories, Inc.
Alexandria, VA 22314, + 1 800-292-4794

For further information, visit www.primolacto.com



Lansinoh Laboratories
Sağlık Gereçleri Tasarım San. Tic. Ltd. Sti.
Zafer SB Mah. Gunduz Sk. No:13 E.S.B
Gazimir/ Izmir/ Türkiye



PL59021IF1023