



Primo-Lacto[®] – Hand Expression Kit


ENFit[®] Compatible

To use while hand expressing:

1. Wash hands with soap and water before each use.
2. Screw an ENFit[®] oral feeding syringe into the bottom of the hand expression funnel.
Ensure a tight fit. Ensure syringe plunger is all the way up.
3. Begin hand expression and collect the expressed colostrum with the hand expression funnel.
4. At the end of the session, pull the syringe plunger down to vacuum the expressed colostrum from the hand expression funnel into the syringe. Remove the syringe.
5. Cap and label the syringe. Follow your healthcare provider's instructions for feeding and storage.

⚠ CAUTION!

Cleaning: Wash with soap and hot water between sessions, rinse with clean water and air dry.

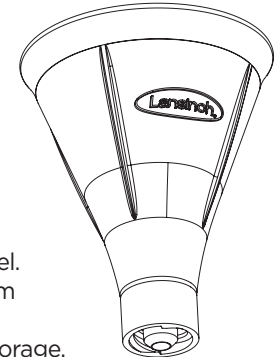


Hand
Expression
Funnel

| | Primo-Lacto [®] |
|----------------|--|
| Before 1st Use | ✓ Sterile – no additional cleaning necessary |
| After Each Use | ✓ Wash – use soap and hot water |
| Once Per Day* | ✓ Sanitize – boil for 5 minutes |

*Only necessary if directed by health care provider

**For hospital use only during the colostrual stage of lactogenesis.
Kit is designed for single patient use only. Components may be
used multiple times. Inspect components prior to each use for
damage or cracks. If required, another kit may be used.**

Lansinoh[®] and Primo-Lacto[®] are registered trademarks of Lansinoh Laboratories, Inc.
ENFit[®] is a registered trademark of Global Enteral Device Supplier Association, Inc.
© 2024 Lansinoh Laboratories, Inc
All rights reserved. Made in China
Distributed by Lansinoh Laboratories, Inc.
Alexandria, VA 22314, +1 800-292-4794
For further information, visit www.primolacto.com

 Lansinoh Laboratories
Sağlık Gereçleri Tasarım San. Tic. Ltd. Şti.
Zafer SB Mah. Gunduz Sk. No:13 E.S.B
Gaziemir/ İzmir/ Türkiye

REF 59020
Catalog number

LOT
Batch code


Date of
Manufacture


Use by date

STERILE EO
Sterilized using
ethylene oxide


Consult
instructions
for use


Manufacturer
name and
address


Keep away
from sunlight


Keep dry


Do not use if
package is
damaged.


Do not
re-sterilize

UDI
Unique device
identifier